

PREGNANCY TESTS

Instructions

1. The BFP Pregnancy Test Strip can be performed at any time of the day; however, testing first morning urine is preferable.
2. Urinate into a clean, dry cup or container.
3. Remove the test strip from the pouch.
4. Hold the test strip in a vertical position with the arrows pointing downward.
Dip the test into the urine approximately 1/4 inch and hold it there for at least 7 seconds.
5. Remove the test strip and lay it down flat. Wait 3 minutes.

Reading Your Results



While you are waiting for your result, you may notice a colored solution moving across the test strip. This is normal. You need to wait 3 minutes to confirm the result. The test should not be evaluated after 10 minutes. The exact color of the line is not important.

1 line = Not Pregnant


2 lines = Pregnant

If only 1 colored line appears as the control line, the test is negative and you can assume you are not pregnant.

If 2 colored lines appear, 1 as the testing line and 1 as the control line, the test is positive and you can assume you are pregnant. Even if the lines are very light in color, you can still assume you are pregnant.

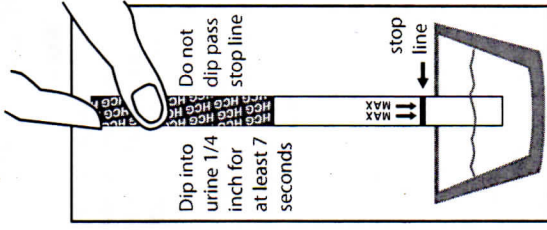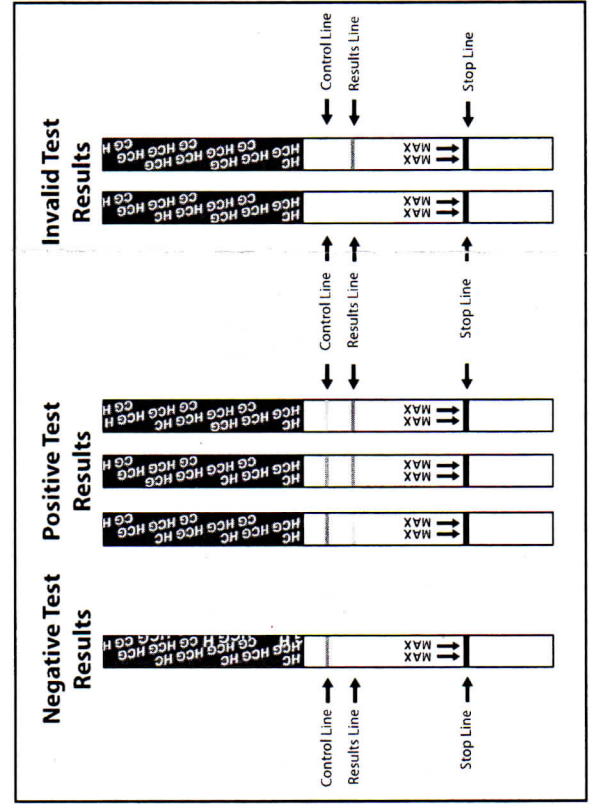

Invalid results: If the control line does not appear, the test is invalid. This can be caused by not holding the test strip in the urine for at least 7 seconds and waiting 3 minutes. Read the results again. If there is still no control line, the test is invalid and you should test again using a new pregnancy test strip. Each test is designed for a single use and should

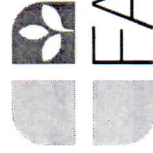

FAIRHAVEN HEALTH

- Ovulation Prediction Tools
- Pregnancy & Ovulation Tests
- Fertility Supplements
- And Much More!

www.fairhavenhealth.com
www.ovagraph.com